


TPC Deluxe Monitor Installation Instructions


1. Remove contents from box.
Remote Control, Monitor, Media Control Box, 50 Pin Data Cable, Power cord for Media Control Box, X-ray Slide Holders.
2. Identify each of the components and place them in desired location.




3. Take the media control box and plug in the power. (Leave Switch In The Off Position). 110V AC from wall.
4. Connect the 50 PIN data cable to the sign connector on the media control box, then connect the other end to the back of the deluxe monitor. This cable allows the monitor to communicate with the Media Control Box.

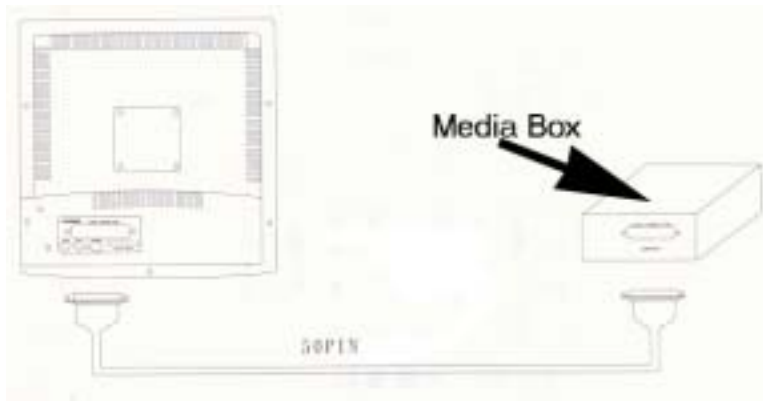


5. To hook up an intra oral camera, install output from camera to RCA input jack labeled Cam in.

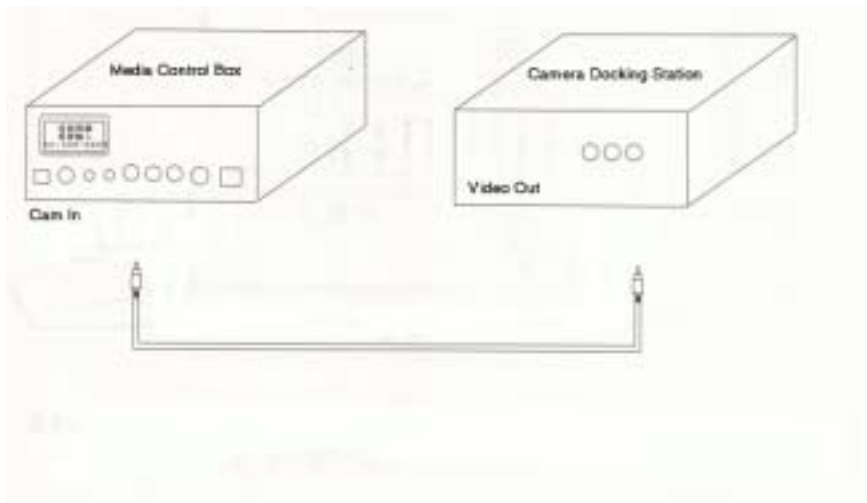

6. To hook up a DVD player or VHS player connect the audio and the video RCA plugs into corresponding inputs).

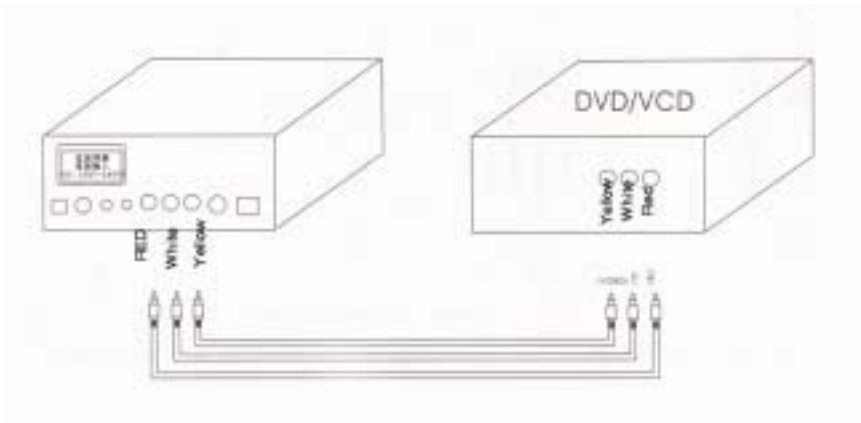


7. To hook up cable TV or an antenna to the media control box locate the TV/CATV input. Connect coax cable from wall to TV/CATV input on media control box.

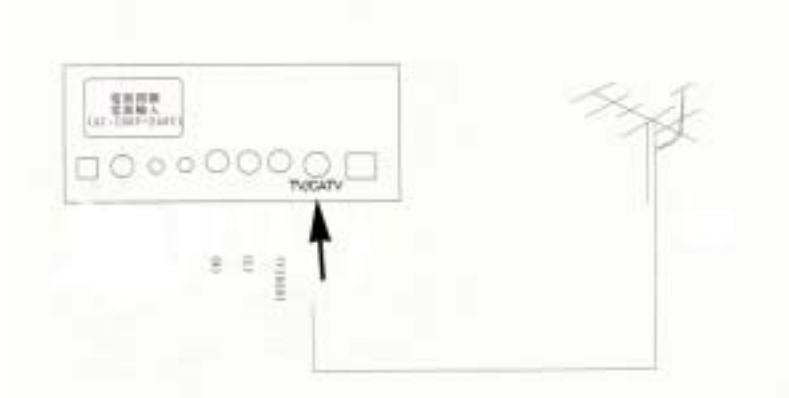


8. Connect your PC to the media control box as follows: Take your VGA out from the computer and connect to VGA “in” on the media control box. Connect your USB cable to the media control box and them to your PC. This will allow you to use the mouse function on the remote control. Follow the plug and play instructions on your computer. Note: If your mouse on the remote does not work you may need to install the mouse driver before the mouse will work; this driver comes with any version of windows.

Note: USB Cable, Audio Cable and VGA Cables are not included.

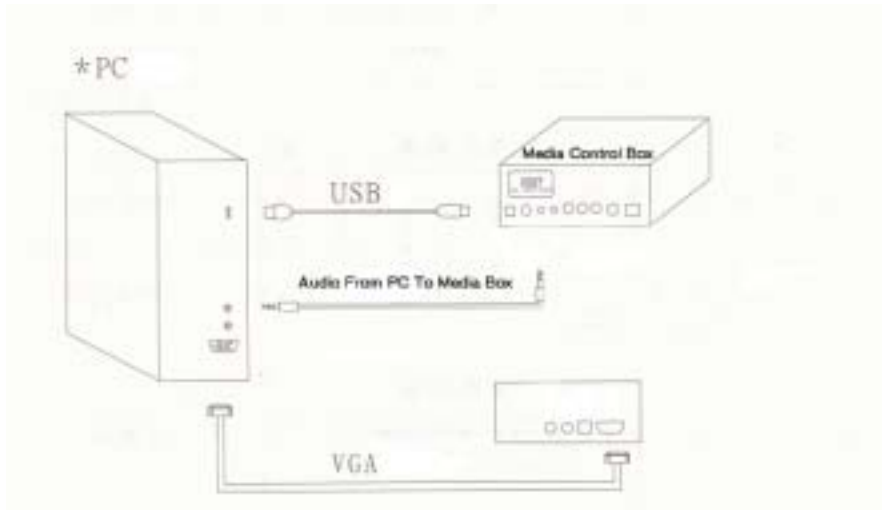


9. The two line ports are used to network the audio and video inputs to an additional deluxe monitor. Note: must use CAT 5 Cable layout.
 Note: CAT 5 Cable not Included.

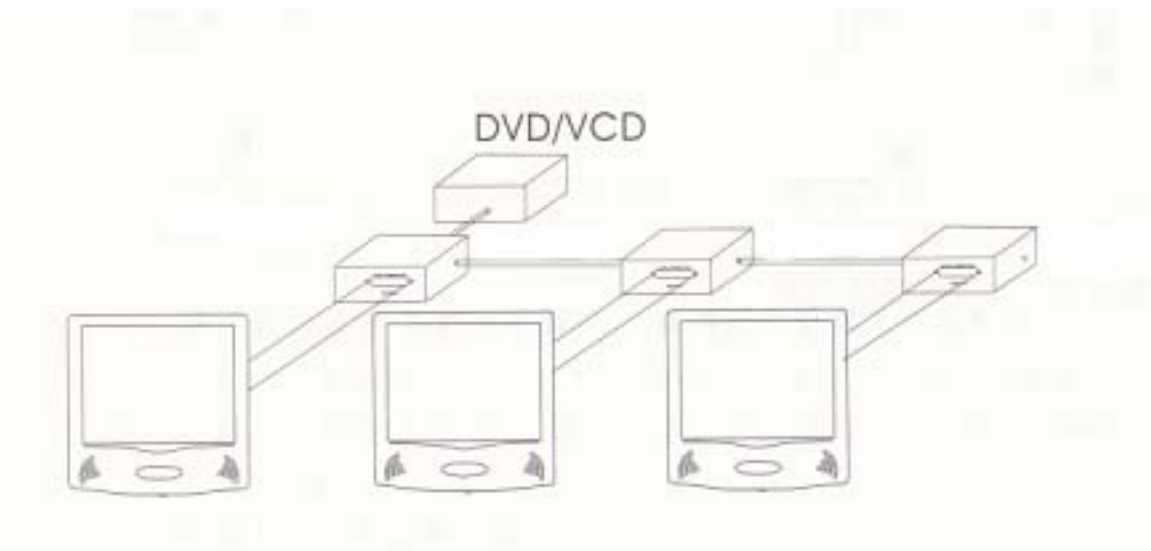

10. The Video “out” from the media control box operates in a way which anything you see on the display will be transferred to the video “out”. This is used to take video from the media box to a computer by way of a video capture card. In turn you can take this image to a computer and save it as a digital file with the assistants of an imaging software program.

Note: Video Capture Card Not Included.

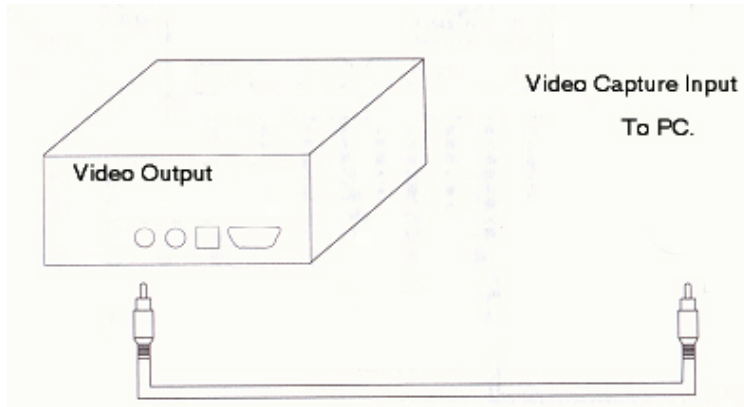

11. You may now turn the power to the ON position on the media control box. After this step is done take the remote control a press the power button, this will turn the power on to the deluxe monitor. Now select the corresponding input on the remote to select which operation you would like to use.

TPC Advanced Technology
City Of Industry, CA
Technical Support
800-560-8222
Techsupport@tpcadvancedtechnology.com

Wireless CAM buttons are Non-functional!!

Power On/OFF

Puts monitor into TV Mode

Volume Adjustment

Puts monitor into AV mode.

Left mouse click for computer use.

Puts monitor into PC Mode.

Will mirror image on screen.

Color adjustment

Contrast adjustment.

Brightness adjustment.

Allows user to change menu settings to the left.



Press to activate laser pointer.

Puts monitor into X-Ray mode.

Changes TV Channel up and down.

LED lights up when buttons are pressed.

Press to change monitor into camera mode.

Right Mouse click for computer use.

Mute.

Color adjustment.

Contrast adjustment.

Brightness adjustment.

Allows user to change menu selection setting to the right.