

Operating Instructions

M-250 Single Speed Amalgamator

***** Attention: Remove (2) bolts from bottom of Amalgamator prior to use!! *****

1. Connect the power cord into outlet receptacle and turn power "On".
2. Open the shield and put the capsule upward into the clips of the amalgamator.
3. Close the cover. The cover must remain closed during operation. If the cover is lifted during operation, the amalgamator will stop automatically.
4. Set mixing time. Please use the table below as a guide that will assist in the determination of an acceptable titrating time for your specific situation. Press start to begin vibration.

<u>Powder or Pellets</u>	<u>Time</u>	<u>Quantity</u>
Metal Pestle	14-16 sec.	One tablet
	16-18 sec.	Two tablet
	18-20 sec.	Three tablet
Plastic Pestle	16-18 sec.	One tablet
	18-20 sec.	Two tablet
	20-22 sec.	Three tablet

Capsulated Alloy

Brands with Pestle	12-14 sec.	Single load
	14-16 sec.	Double load
	16-18 sec.	Triple load
Brands without Pestle	14-16 sec.	Single load
	16-18 sec.	Double load
	18-20 sec.	Triple load

Variations between machines and line voltage may affect mixing time for your application. These suggested times are to be used as a guide only.

****** CAUTION: RISK OF INJURY IS POSSIBLE IF THE COVER IS NOT CLOSED PRIOR TO OPERATION !! *****

