

TPC WXR-700 Wireless X-Ray Viewer

Installation for Corded Use



1. Remove contents from packaging.
2. Plug in the 12 Volt DC power Supply into the back of the WXR-700.
3. Plug in the yellow RCA connector into the video output of the WXR-700.
4. Connect the corresponding end of the RCA cable to the input of the display adaptor. (TV, Video Capture Device, LCD Monitor).
5. Turn on the power to the WXR-700 by pressing the power button on the front of the unit.



6. Pull door open to load film, Push the film door to close and you will see the X-Ray Film projected onto your display device.

Installation For Wireless Use



1. Remove contents from packaging.
2. Plug in the 12 Volt DC power supply.
3. Connect your receiver to your display device via the RCA cable supplied with the WXR-700. This cable will run from the output of the transmitter to the input on the display device. Also plug in the 5 Volt power supply to the transmitter.



4. Select the desired wireless frequency on the receiver by pressing the choice button.



5. Turn on the WXR-700 X-Ray film reader via the power button on the front of the device. (It will Default to Channel 1)



6. To Change Channels push the CH Select button located on the back of the WXR-700. this will allow the user to change from CH 1 to Ch 2 to CH 3 to Ch 4.
NOTE: the channels must match (WXR 700 XR 745 XR 750) or the wireless mode will not function correctly.
7. The X-Ray / Video function on the back of the WXR-700 will allow the user to input a signal such as the wireless intra oral camera through the WXR-700 unit.

