

# Advance 750

Piezo Scaler

## USER'S MANUAL



**CE**  
0434

- 
- The device must not be used when the patient and / or the operator has a cardiac pacemaker.
  - Read all instructions in this USER'S MANUAL BEFORE installation or operation.

# TABLE OF CONTENTS

|   |          |
|---|----------|
| <b>1. PRESENTATION</b>                  | <b>1</b> |
| <b>2. ADVANTAGES</b>                    | <b>1</b> |
| <b>3. CONSTITUTION</b>                  | <b>1</b> |
| <b>4. INSTALLATION</b>                  | <b>2</b> |
| <b>5. OPERATION</b>                     | <b>3</b> |
| <b>6. MAINTENANCE</b>                   | <b>5</b> |
| <b>7. TROUBLE SHOOTING</b>              | <b>5</b> |
| <b>8. SPECIFICATION AND ACCESSORIES</b> | <b>6</b> |
| <b>9. WARRANTY</b>                      | <b>7</b> |

## NOTE

Use only the specified accessories.



: Read before use.



: Type BF applied part.





: Double insulated equipment.

# 1. PRESENTATION

The Advance750 Piezo Scaler is designed for the dentists to remove the calculus or stains on surface of teeth or clean the root canal (with Endo-kit) in prophylaxis procedures.

- 1》 Handpiece is autoclavable.
- 2》 High frequency and low amplitude brings patients more comfort.
- 3》 Ti-Handpiece produces less heat and is suitable for long time operation.
- 4》 Efficient performance ensures calculus removal.
- 5》 Easy control of power and water spray.
- 6》 Endo kit is an option.
- 7》 Minimum dentin removal keeps the integrity of tooth structure.

# 3. CONSTITUTION

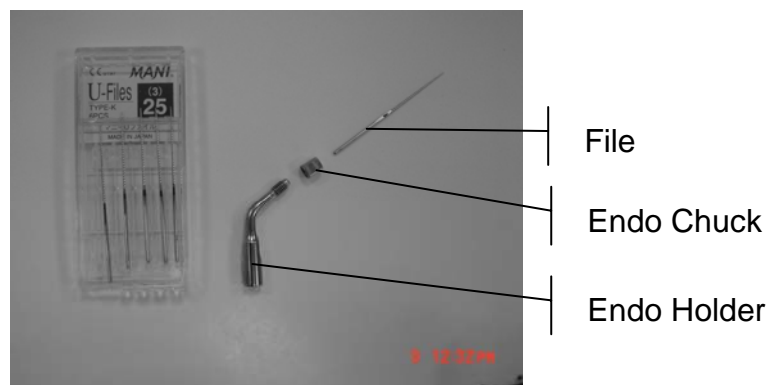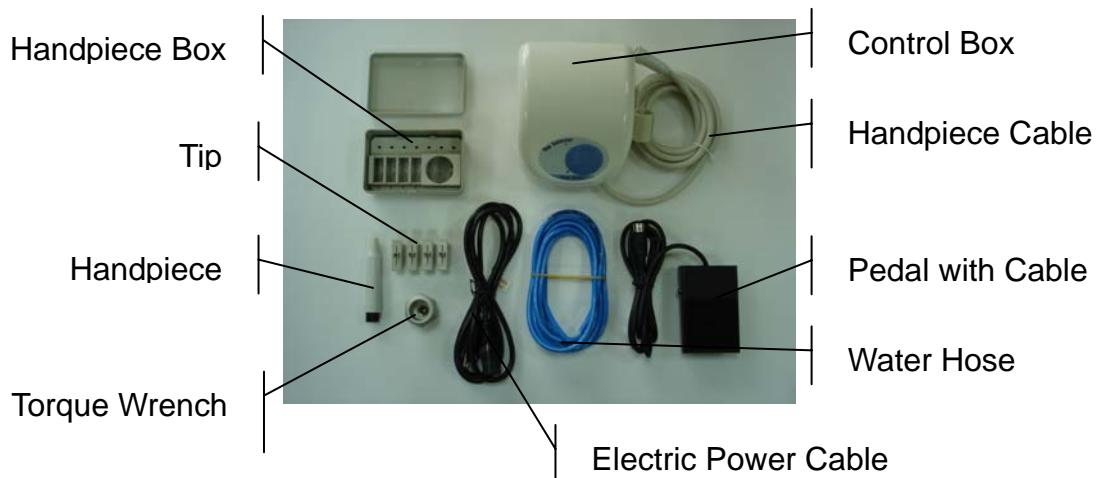

## 4. INSTALLATION

- 1》 Install the water line to sufficient water source.  
\* **Water is a must for ultrasonic scaler machine.**
- 2》 Connect the foot pedal to the foot pedal switch properly.
- 3》 Install Scaling tips.

**Step 1** Place tip onto handpiece first, then carefully turn it to be a little tight.



**Step 2** Use torque wrench to tighten the tip firmly till “click” sound is heard.



**Step 3** To prevent self-locking, the Tip should be dismantled once a month.

- 4》 Install Endo Holder & File.

**Step 1** Use torque wrench to affix the holder on handpiece tightly.











