

*AdvanceCAM
IntraOral Camera
Operation And
Installation Manual*



**Models covered in this manual are as follows.
AIC-899 USB Camera**

Installation Instructions for AIC899 Camera

1. Place the software disc into the computer drive.
2. When menu appears press install TPC soft image icon once. Follow the on screen instructions, then click finish.
3. Now press the install AIC899 driver. Follow the on screen instructions. Make sure that you select the USB DK video driver, USB audio driver and BG video driver. Then press next. The drivers will install automatically.
4. When the driver signature appears, click “continue anyway”. You will have to do this twice.
5. Once the installation is complete you must restart your computer.
6. Once the computer restarts plug in you camera like the diagram shown to the right. Follow the on screen instructions.
7. Select install software automatically. Then Click next. When prompted to select “continue anyway” when asked to by the software wizard. On some computers you may have to restart an additional time. Now proceed to use the TPC soft image icon on you computer to use your camera. Refer to the TPC soft image located on you software disc to review the operation of the TPC soft image program.

Installation Instructions for AIC899 Camera



Warning:
Always replace disposal sheath after each patient to prevent cross contamination. Always apply sterilization on camera prior to replacing new sheath.




Freeze: Press the freeze button to capture the image into TPC soft image program. Capture windows must be open and a live image must be present to perform this function.

Full / Quad: Press this button to turn the camera on. Press and hold again for 5 seconds to turn the camera off.



Connect USB Cable to camera like shown in the picture shown to your left.

Connect the other end of the Camera to your USB port on the computer.

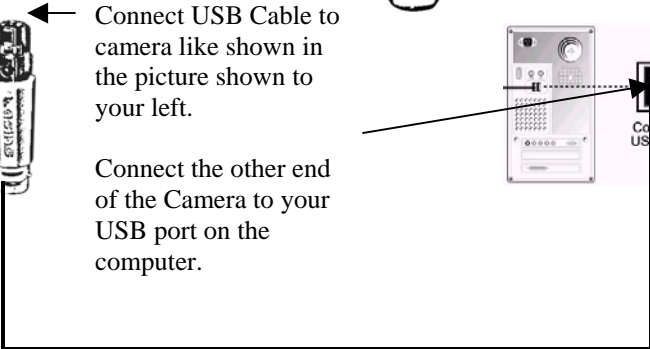

Warranty: TPC warranties most camera components to be free from defect for one year. Components not covered under warranty are as follows: LED's are rated at 100,000 hours but not under warranty for any period. Batteries are excluded from this warranty as well. This warranty excludes negligence on behalf of the end user. Most camera components can be repaired, if an item can't be repaired and it's under warranty, TPC will replace the item.

TPC Advanced Technology

18525 E. Gale Ave

City Of Industry, CA 91748

Tel: 800-560-8222

Fax: 626-810-4245

www.tpcdental.com

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

Name:

Address:

Telephone # _____

Product Serial Numbers:

Camera wand _____

Docking Station _____

Transmitter _____

Include a copy of you original purchase invoice