

TPC

X-Ray Chair Installation



TPC

851 S. Lawson St.

City Of Industry, CA 91748

P626-810-4337 Fax 626-810-4245

1

www.tpcdental.com

REV. 1-1-13

TPC

Note! Your new X-ray chair will be packaged in three boxes. Make sure that you have all three boxes before beginning.

1. Remove the base from its box. Place it in an open area close to the desired installation location.



2. Open the remaining two boxes. Locate the six chair cylinder bolts and lock washers then place the chair cylinder in the base as shown.



3. Once the bolts are in place, turn the base over and locate the foot lever. Remove the set screw and install the foot lever as shown. Make sure that the lever is angled up as shown. Once installed tighten the set screws.



4. Lay the base down on its side and locate the chair seat cushion. Insert the seat cushion into the cylinder. Note the notch in the bracket and the pin located in the cylinder.



TPC

5. Turn the stool upside down so that it's resting on the seat cushion. Install the foot rest as shown. Use the 4 Allen screws and fasten them securely.



6. Turn the stool on its side and begin to install the armrests. Use the 6 bolts and washer sets to firmly fasten them in place.



7. Locate the backrest cushion and attach it to the seat cushion. Use the included bolts and washer set to attach them.



8. Loosen the adjustment knob on the headrest stem assembly to slide the headrest stem into place. Once the headrest stem is in place you may tighten the adjustment knob.

