

# Mirage Model 4000 Hydraulic Patient Chair



## Programming preset positions on foot control

TPC


851 S. Lawson St

City Of Industry, CA 91748

P626-810-4337 Fax 626-810-4245

[www.tpcdental.com](http://www.tpcdental.com)

1. Make sure the dental chair is in operational mode by moving the chair base up and the backrest back. Then press the auto return button "0". The chair base should go down and the backrest should come up. This will put the chair in an exit position for the patient.
2. To program a preset positions do the following.
3. Move the chair to the desired position by using the manual disc.

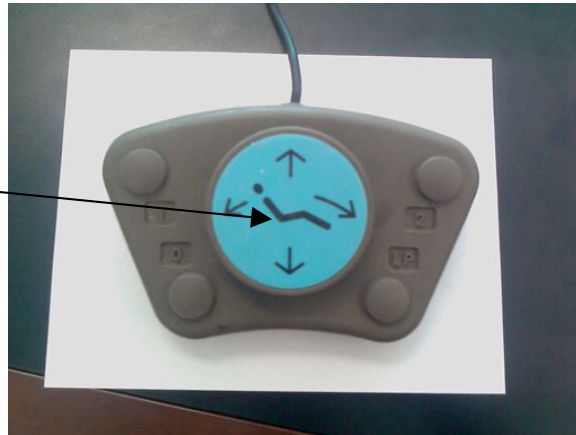

4. Once the chair is in the desired position, Press and hold the programmable button that you want the position programmed to. Either 1 or 2. At first the chair board will beep when the button is depressed. 3 – 5 seconds later you will hear another beep letting you know that the position has been programmed.



5. You have now completed programming the programmable position on the foot control.
6. You may not program position 0 or LP. 0 is auto return and is determined by the chairs lower base limit and backrest upper limit.