



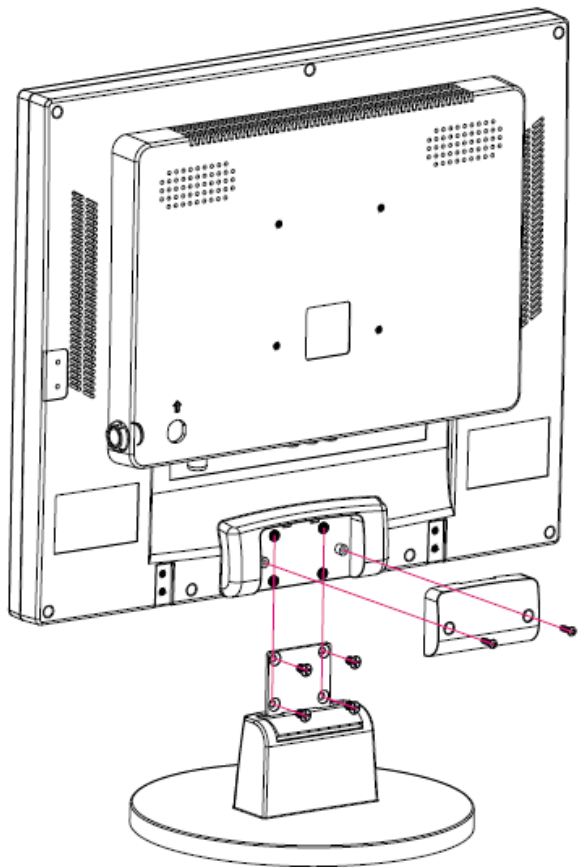

17-Inch Multi-Media Monitor Display
Accessories Assembly Diagram

Owners Manual

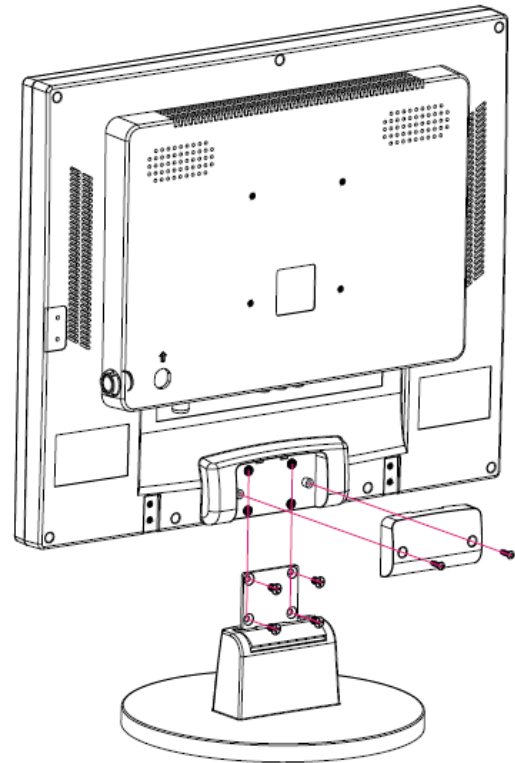

TPC

17— Inch Multi-Media Monitor

Manual

Part Number: #MO17

Manual

Table Of Contents

1. Product Introduction	1
2. Function & Description	2
1. Multimedia Display	2
2. Remote Control	3
3. Troubleshooting	4
4. Precautions	4
5. Storage & Transport	5
6. Warranty Instructions	5

Manual

5. Storage & Transport

- Please be gentle when removing and placing video input cables.
- The Monitor is not:
 - Fire Resistance - plastics will melt and warp with excessive heat
 - Impervious to Corrosive Solutions - plastic will warp when exposed to harsh chemicals
- Storage temperature relative humidity indoors at or below 93%
- Store in ambient pressure, between 50Kpa & 106Kpa
- Avoid excessive vibrations and jolts during transport.
- Avoid direct sunlight, rain, or snow exposure during transport.

6. Warranty Instructions

The MO17 comes with a 2-year limited warranty. Under standard TPC Terms & Conditions, a user has 7-day from the delivery of the unit to return for a full refund. Shipping is not covered under warranty. If you experience any difficulties or need help troubleshooting the unit please call our technical support line at 800-560-8222; otherwise please visit our website for additional information: <http://www.tpcdental.com/terms.html>

Manual

3. Troubleshooting

➤If you experience any issues when operating your MO17 monitor please refer to the following chart; if you are still experiencing any issues please call our technical support line at 800-560-8222

Item	Issue	Solution
1	The monitor does not turn on	Check to see that the monitor power cord is plugged and power switch is turned on
2	Permanent (black) spots on image	Check to see if the screen is stained, check to see if the camera lens is tainted; if it is, clean gently with clean cotton swab to wipe debris away

4. Precautions

- Please read the following precautions before using this product, and keep for future reference.
- Do not use sharp objects, corrosive solution or abrasive materials on any electrical contacts.
 - Please unplug monitor and camera when not in use.
 - Do no disassemble any portion of the Monitor.
 - Do not use force when connecting components or cables.
 - Use the original components such as the power adapter and cord.
 - Do not use, if any of the components is damaged or broken; please order original manufacturer components from an authorized TPC dealer.
 - Do not force any power cord or power adapter into position.
 - Do not use a wet power cord or adapter.
 - Use MO17 monitor in a well ventilated, dry place.
 - Monitor will not work in extreme cold, hot or humid conditions, do not place in direct sunlight or dusty environment.

Manual

1. Product Introduction

Congratulations on your new TPC MO17 monitor & intra-oral camera solution purchase. To get the maximum benefit from your monitor, read these instructions before making any adjustments, and retain them for future reference. We hope you will experience many years of enjoyment from your new monitor.

For further assistance, please call: 800-560-8222 or visit our website: www.tpcdental.com

Purchase Date: _____

From Authorized Dealer: _____

2. Product Descriptions






3. Product Features & Technical Specifications

Display Mode	Single, Four
Resolution	1280 x 1024
System Operations	Touch, Remote Control
Ambient Temperature	32° - 131°F
Power Consumption	<35W

2. Function & Description

1. Multimedia Display

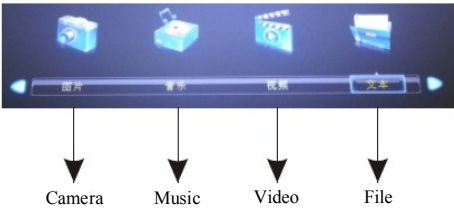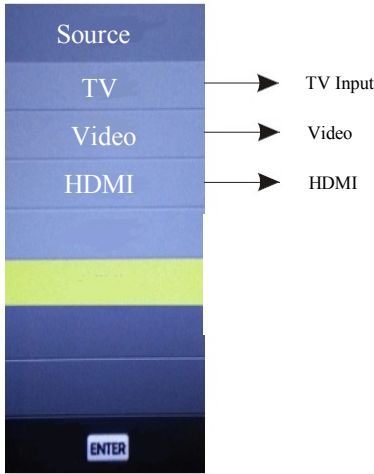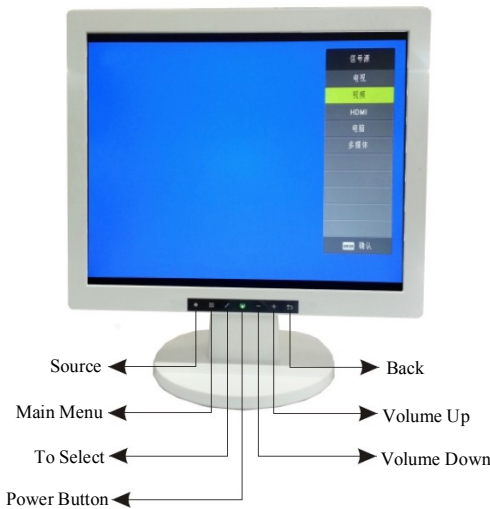

3. Multimedia Display Remote Control

