




Superb Vac Parts List WV6

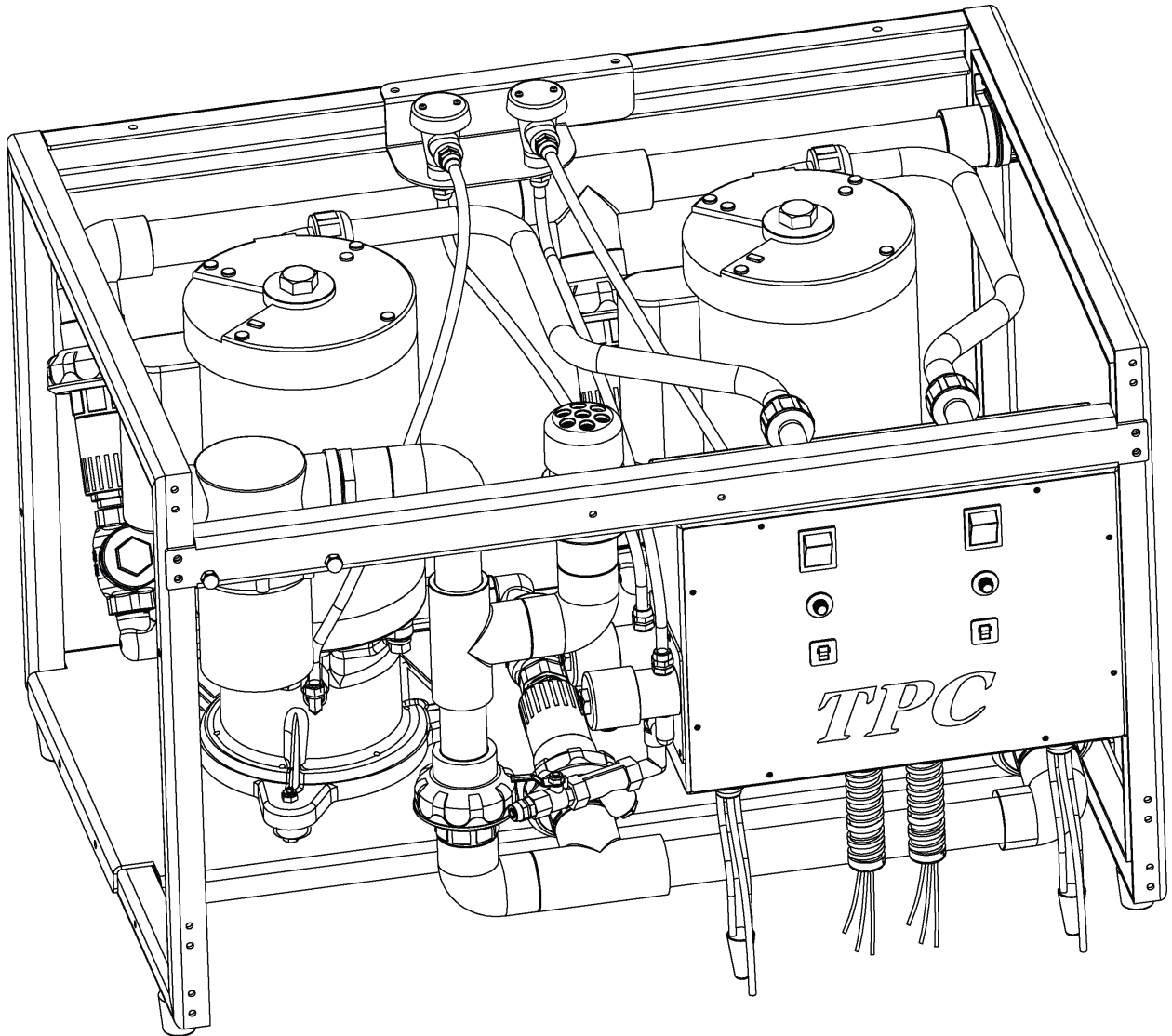

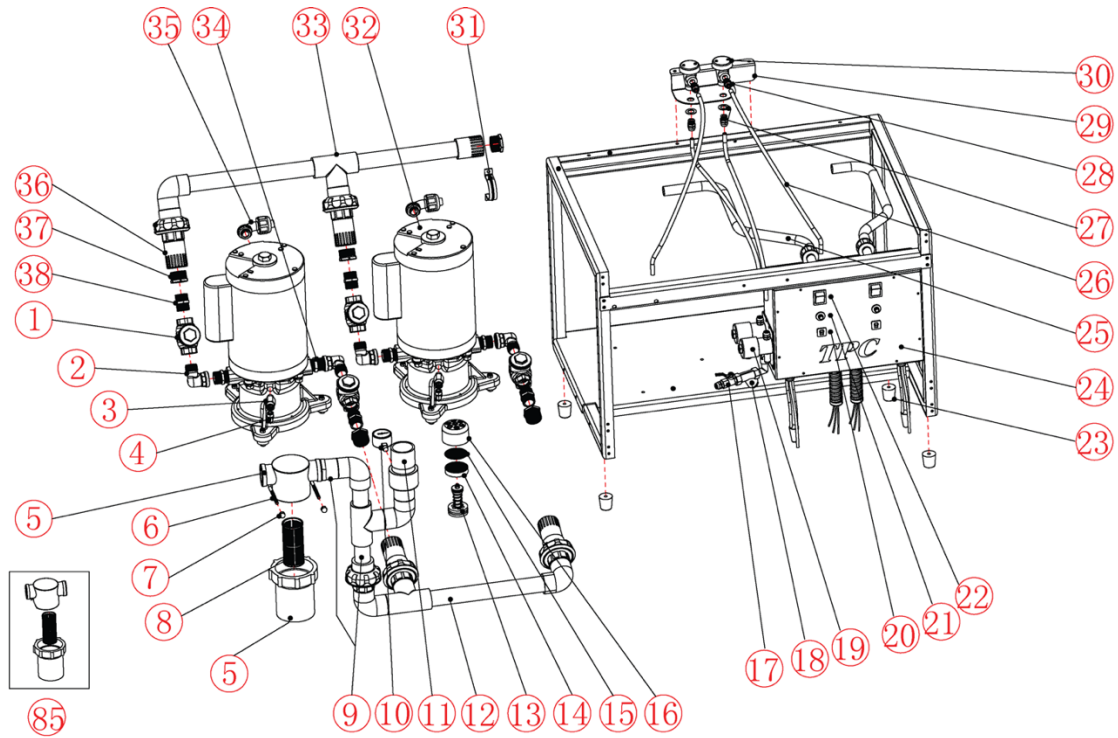


No.	Description	Part Number	No.	Description	Part Number
1	Check Valve 3/4"	WV6-RP-1	21	Resettable Fuse 20 Amp	WV6-RP-21
2	Elbow 3/4"	WV6-RP-2	22	Main Power Switch	WV6-RP-22
3	Water Check Valve	WV6-RP-3	23	Rubber Cage Legs X4	WV6-RP-23
4	Poly Coupler	WV6-RP-4	24	Control Panel face	WV6-RP-24
5	1" Vacuum Filter Clear Bowl Assembly	WV6-RP-5	25	Flex Conduit	WV6-RP-25
6	U Bracket	WV6-RP-6	26	6 x 4 Water line "Black"	WV6-RP-26
7	U Bracket Nut	WV6-RP-7	27	Poly Coupler	WV6-RP-27
8	Filter Screen for 1"	WV6-RP-8	28	Poly Coupler	WV6-RP-28
9	Suction Manifold 1	WV6-RP-9	29	Vacuum Breaker Bracket	WV6-RP-29
10	HG Gauge	WV6-RP-10	30	Vacuum Breaker	WV6-RP-30
11	Pressure Relief Body	WV6-RP-11	31	1" manifold Clamp	WV6-RP-31
12	Suction Manifold 2	WV6-RP-12	32	230V Vacuum Motor	WV6-RP-32
13	Pressure Relief Valve	WV6-RP-13	33	Exhaust Manifold	WV6-RP-33
14	Pressure Relief Baffle	WV6-RP-14	34	3/4" Union	WV6-RP-34
15	Pressure Relief Screen	WV6-RP-15	35	Conduit Cord Grip	WV6-RP-35
16	Pressure Relief Cap	WV6-RP-16	36	Exhaust Manifold Coupler	WV6-RP-36
17	Water Angle stop	WV6-RP-17	37	Union Reducer 1" X 3/4"	WV6-RP-37
18	Water filter Assy	WV6-RP-18	38	Union 3/4"	WV6-RP-38
19	220V Solenoid	WV6-RP-19	85	1" Filter Assembly Complete	WV6-RP-39
20	Breaker 0.5 Amp	WV6-RP-20			

WV6 Control Panel Parts List

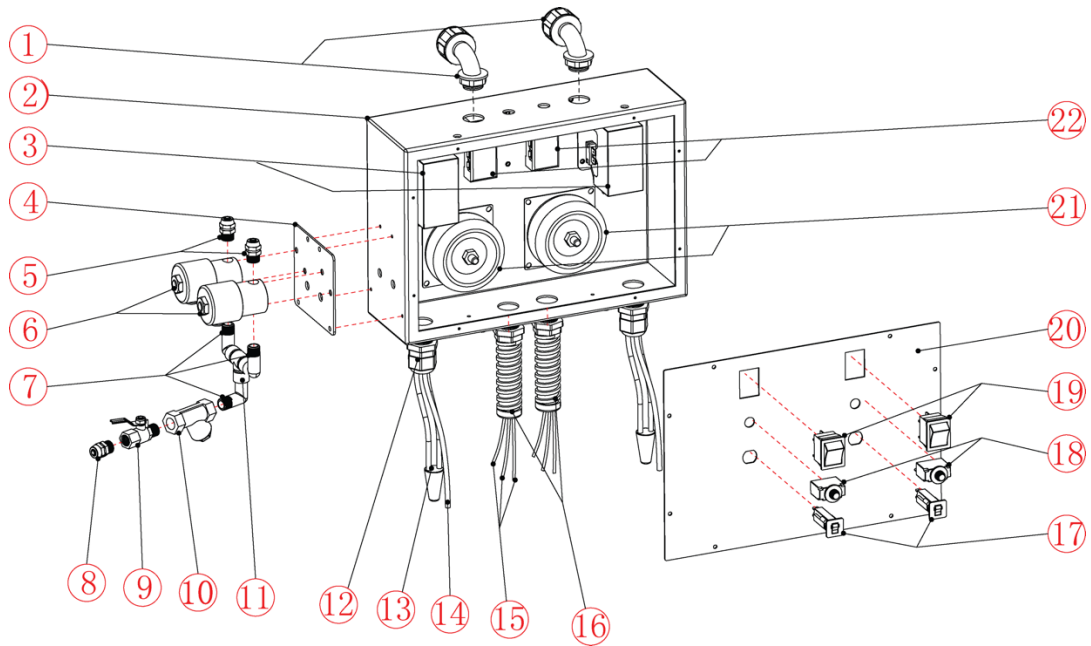

No.	Description	Part Number
1	Conduit Cord Grip	WV6-CP-RP-1
2	Control Box	WV6-CP-RP-2
3	CHINT Contactor NCK3-32/2	WV6-CP-RP-3
4	Solenoid Mounting plate	WV6-CP-RP-4
5	Poly Coupler	WV6-CP-RP-5
6	220V Water Solenoid	WV6-CP-RP-6
7	Water line Y Union	WV6-CP-RP-7
8	Poly Coupler	WV6-CP-RP-8
9	Water Angle stop	WV6-CP-RP-9
10	Water filter Assy	WV6-CP-RP-10
11	90 Degree Union	WV6-CP-RP-11
12	Straight Cord Grip	WV6-CP-RP-12
13	Wire Nut	WV6-CP-RP-13
14	Wire	WV6-CP-RP-14
15	Motor Whip Voltage line	WV6-CP-RP-15
16	Conduit	WV6-CP-RP-16
17	Breaker	WV6-CP-RP-17
18	Resettable Fuse	WV6-CP-RP-18
19	Power Switch	WV6-CP-RP-19
20	Panel Cover	WV6-CP-RP-20
21	24 V AC Transformer	WV6-CP-RP-21
22	4 X 4 Terminal Block	WV6-CP-RP-22

WV6 220V Motor Parts List

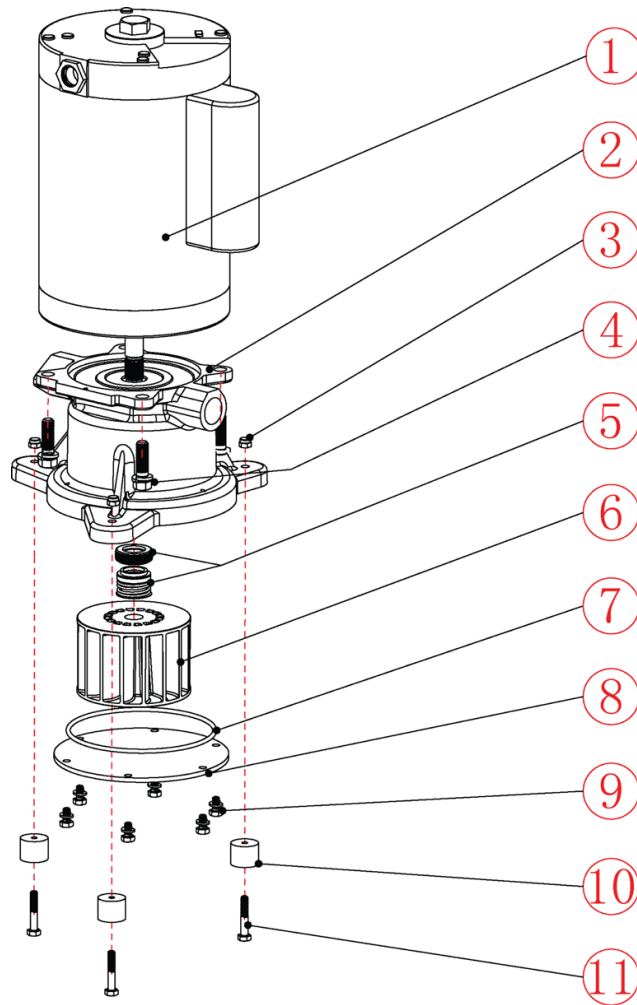

No.	Description	Part Number
1	Motor Case	NA
2	Inlet Plate	NA
3	Nut	NA
4	Bolt	NA
5	Bearing / Bushing	NA
6	Impeller	NA
7	O-ring	NA
8	Base Plate	NA
9	Screws	NA
10	Rubber Legs	NA
11	Leg Bolts	NA
12	Complete Motor	